

MATRIX™ Manufacturer's Instructions

Thank you for choosing The PostureWorks MATRIX™ wheelchair cushion. Please familiarize yourself with the proper use and care of your new PostureWorks device. If you have questions, or need further assistance, please contact us toll free at 1-866-860-5200; a representative will personally assist you.

All PostureWorks products should be used and purchased under the supervision of a qualified healthcare provider; these products are not a substitution for proper patient supervision and care.

INSTALLATION: The MATRIX wheelchair cushion must be the same width and depth as the sling seat wheelchair upon which it will rest. Place the MATRIX wheelchair cushion directly on top of the existing sling seat so that both of the lateral sides of the cushion are evenly placed in the wheelchair.

The MATRIX is manufactured with a strap located at the rear of the device. This strap helps secure the cushion to the wheelchair and prevent it from sliding off the front of the chair under normal use. Wrap the MATRIX strap around the back of the wheelchair, fasten the buckles to secure the strap, and tighten the strap so the device is fitted tightly to the wheelchair.

The MATRIX is manufactured in various sizes to fit standard size wheelchairs. The MATRIX should fit securely on top of the sling seat wheelchair with which it will be used. If your PostureWorks device does not fit securely in the wheelchair, you may need a custom size device. To order a custom size device, please call The PostureWorks toll free at 1-866-860-5200.

ACCESSORIES: To prolong the life of your MATRIX wheelchair cushion, The PostureWorks recommends using Dycem strips, or a similar barrier to cover any exposed bolts on the sling seat of the wheelchair. Exposed bolts on wheelchairs can snag the material on an orthotic wheelchair cushion and cause it to rip/tear. To order Dycem strips, please call 1-800-458-0060.

CLEANING: The MATRIX can be cleaned using a mild detergent or water and wiped clean with a damp cloth. The device can be dried with a soft dry cloth. This product should be kept clean and dry. **DO NOT IMMERSE OR SOAK IN LIQUID.**

The PostureWorks <> Toll Free number 1-866-860-5200 <> Fax 781-431-8202
www.ThePostureWorks.com

April 2014

When cleaning the wheelchair, the MATRIX should be removed from the wheelchair prior to cleaning. The MATRIX **SHOULD NOT BE EXPOSED TO RUNNING WATER OR LIQUID UNDER PRESSURE.**

USAGE: The MATRIX™ wheelchair cushion must be the same width and depth as the seating surface upon which it will rest. Safety straps should be attached to the rear of the wheelchair. The PostureWorks MATRIX™ wheelchair cushion should always be fitted and monitored by a qualified healthcare professional.

Installation of a wheelchair cushion may affect the center of gravity of the wheelchair and cause the chair to tip backwards, resulting in injury for the occupant. Always **EVALUATE THE REQUIREMENT FOR WHEELCHAIR ANTI-TIPPERS** or other adaptive devices to ensure proper wheelchair stability.

WARNING: The dry gel insert of the PostureWorks MATRIX™ wheelchair cushion is covered with a fire barrier sock to comply with CA Fire Bulletin 133. **DO NOT REMOVE FIRE BARRIER SOCK.**

WARRANTY: The MATRIX insert is warranted to be free from defects in materials and workmanship for a period of 5 years from date of purchase under normal use.

Should a defect occur, The PostureWorks will, at its sole option, repair or replace the device without charge. This warranty does not apply to punctures, tears, or burns.

Except for express warranties made herein, all other warranties, including implied warranties or merchantability and warranties of fitness for a particular purpose are excluded.

There are not warranties which extend beyond the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of the non-conforming goods sold.

The PostureWorks <> Toll Free number 1-866-860-5200 <> Fax 781-431-8202
www.ThePostureWorks.com

April 2014